

UVone™

Disinfects mobile devices to a 5-log* kill in just 30 seconds, reducing the risk of hospital-acquired infections.



External Specs: 381 mm (L) x 114.3 mm (W) x 304.8 mm (H) | 4.8 kg
Internal Dimensions: 273.05 mm (L) x 38.1 mm (W) x 190.5 mm (H)
Fits Devices up to: 271.52 mm (L) x 38.1 mm (W) x 188.97 mm (H)

Reduce the Spread of Infection



99.999* Bacteria Reduction. Effective UV-C light disinfects to a 5-log kill, reducing the colony to 10 MRSA bacterium after a 99.999%* reduction. UVone reduces SARS-CoV-2, the virus that causes COVID-19, by 99.99%*—which is 10x more effective than leading disinfecting wipes. With touchless sensors, users never physically touch the station, reducing cross contamination.



Rapid UV-C Technology Disinfects in just 30 Seconds. Device sanitisation for healthcare workers continues to be an on-going issue for hospitals trying to reduce the spread of HAIs. UVone is specifically designed for quickness and ease of use, encouraging busy healthcare professionals to change their daily behavior, thus increasing hygiene compliance long term.



Six Times Quicker Than Traditional Wipes. To be effective, traditional wipes can take upwards of three minutes to disinfect, while UVone disinfects in just 30 seconds. UV-C disinfection is more reliable and efficient than wipes—plus, it does not degrade the integrity of devices over time.



Convenient and Visible. UVone seamlessly integrates into any environment or workflow, such as outside of operating rooms, patient rooms, nurses' stations, or any other location that is most convenient and visible for compliance. A VESA-compatible mounting plate and wall mount are included with every UVone Station. A VESA-compatible table stand or cart with wheels (sold separately) are available upon request.


*LocknCharge's kill rate claims are based on data conducted by accredited, BSL-compliant laboratories. For more information about our testing data, please visit www.lockncharge.com/eu/uvone-testing-data.

Simple and fast operation increases hygiene compliance.

Reduce the spread of hospital-acquired infections in just 30 seconds by following these simple steps.




To open, wave hand above station.

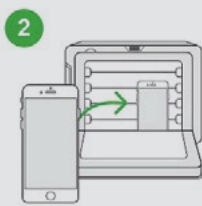

Place your device inside so it rests on the glass.



Wave hand above station to close, and wave again to begin cycle.

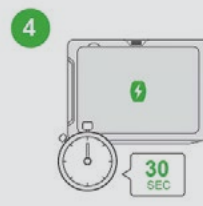

The display indicates your device is being disinfected.



UVone will open once your device is disinfected.



To close, wave hand above station.

Additional Key Benefits



Compatible with Almost Any Mobile Device. UVone can accommodate any device within 271.52 mm (L) x 38.1 mm (W) x 188.97 mm (H) –including tablets, iPad devices, iPhones, Vocera phones, Voalte phones, Ascum phones and more, with or without a case.¹ It can disinfect up to three smaller devices at one time or even items like keys and badges.



Form Meets Function. The sleek, compact design of UVone seamlessly integrates into any environment. Mount the station directly outside of operating rooms, patient rooms, nurses' stations or any area that is most convenient for use. The station is ideal for critical care areas such as NICU, SCBU and ICU.



Countdown Timer. Users are able to see how much time remains until contents are disinfected.



Touchless, Motion-Activated Door. Most HAIs are transmitted by physical contact, making personal hygiene of hospital staff a critical step toward reducing the spread of infection. UVone users never physically touch the station, preventing cross contamination.



Ready at All Times. The UVone station is easy to use. No need to turn off your device before use.



World-Class Customer Support. Our helpful and friendly Customer Support team boasts an NPS score higher than many tech giants and will promptly assist with any product issues or questions that may arise



25,000 Hours (3 million uses) of Bulb Life. 16 long-lasting UV-C bulbs are enclosed behind glass. When used in Automatic Cycle Mode, users can expect 25,000 hours of bulb life or 3 million uses.



Easy-to-Use App. In the event a bulb burns out, the app provides visibility to see which bulb needs to be changed. From the app, users can switch a station from Manual Mode to Automatic Mode and update firmware.



1-Year Warranty.² UVone is backed by a 1-year warranty, protecting you against product defects.



Thinking Beyond Sanitisation. LocknCharge offers a complete suite of solutions. Easily pair UVone with a LocknCharge charging station that works best for you to ensure devices are reliably sanitised, charged and ready for use at all times.



www.lockncharge.com/eu
Phone: +44 (0) 208 441 4107
europesales@lockncharge.com



lockncharge™

Specifications and images are for illustration purposes only. Final product may differ. iPad and MacBook are trademarks of Apple Inc., registered in the U.S. and other countries. Lightning, iPad Air and iPad mini are trademarks of Apple Inc. *LocknCharge's kill rate claims are based on data conducted by accredited, BSL-compliant laboratories. For more information about our testing data, please visit www.lockncharge.com/eu/uvone-testing-data. Real world results may vary. Testing data available upon request. ¹Compatible cases must be non-porous and remain within dimensions of 271.52 mm (L) x 38.1 mm (W) x 188.97 mm (H). ²Visit www.lockncharge.com/eu/warranty for complete warranty details.

